



TREATING ANIMALS CALLS FOR AN EXTRA HAND

Whether you're a Veterinarian or Livestock Producer it's time to make your workload easier!

*Maintain use of both hands when working animals by safely holstering a capped, single-dose syringe in your **EZ CAPPER SYRINGE HOLSTER**.*



*Patent Pending

WHEN YOU'RE READY TO HAVE QUICK, EASY, AND SAFE ACCESS TO YOUR SYRINGE WHILE ADMINISTERING INJECTIONS:

- ▶ Simply pull syringe straight out, this will uncap syringe while leaving the cap in the holster spring.
- ▶ Once injection is complete, simply re-cap syringe by inserting needle back into the cap resting in the holster; using enough force to press the cap onto the needle.
- ▶ To safely remove capped syringe from holster, push the end of the syringe away from wrist to release syringe and capped needle from the spring.

Product # 9080 - EZ Capper Syringe Holster*



Visit Stonemfg.net
to find your authorized distributor.



QUALITY • SERVICE • INNOVATION
1212 Kansas Ave. Kansas City, MO 64127
(816) 231-4020 • Fax (816) 241-3336
www.stonemfg.net